

The fast-setting plaster
of Paris bandage
with an extremely short
immersion time:




Plastrona®

Plastrona is a fast-setting plaster of Paris bandage whose special properties are due to the ideal composition of the plaster mass and the upper and porous impregnation of plaster on the support material.

Plastrona bandages are available for circular plaster of Paris bandaging techniques, Plastrona Slabs are used as splinting bandages and to provide additional strength applied on top of circular plaster of Paris bandages, and Plastrona Large Slabs are used for large plaster casts.

Description

► Due to its porous structure, Plastrona is moistened rapidly and uniformly. Immersion time is only approx. 2 – 3 seconds, which means that Plastrona may be worked with rapidly.

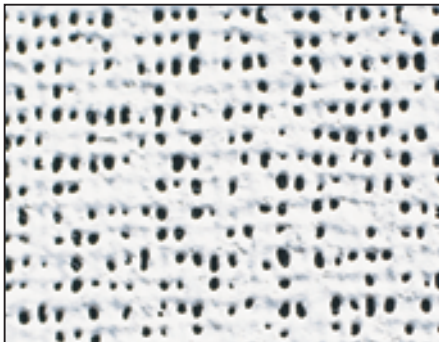

Structure of Plastrona fast-setting plaster of Paris bandage, magnified 4 1/2 times.

► The mixture of high-quality plaster types affords Plastrona good thickness and excellent moulding ability.

► Plastrona has a moulding time of 2 – 3 minutes and sets after approx. 5 minutes at an immersion temperature of 20 °C.



The extremely short immersion time of Plastrona has a decisive influence on the quality and stability of the applied plaster of Paris bandage. No dry bandage areas occur which could affect the final strength, nor is excessive water absorbed which would disturb intertwining of the plaster crystals on setting and delay drying of the bandage.

Presentations

		Unit	Code No.	Units per Case
<u>Plastrona Bandages</u>				
individually sealed 2.75 m length	6 cm	12 pieces	332 390	12
	8 cm	12 pieces	332 391	10
	10 cm	12 pieces	332 392	8
	12.5 cm	12 pieces	332 393	6
	15 cm	12 pieces	332 394	6
	20 cm	12 pieces	332 395	4
sealed in packs of 2 bandages 2 m length	6 cm	5x2 bandages	332 313	10
	8 cm	5x2 bandages	332 314	8
	10 cm	5x2 bandages	332 315	12
	12 cm	5x2 bandages	332 316	10
	15 cm	5x2 bandages	332 317	9
	20 cm	5x2 bandages	332 318	6
3 m length	8 cm	5x2 bandages	332 334	10
	10 cm	5x2 bandages	332 335	8
	12 cm	5x2 bandages	332 336	8
	15 cm	5x2 bandages	332 337	7
	20 cm	5x2 bandages	332 338	4
4 m length	8 cm	5x2 bandages	332 344	10
	10 cm	5x2 bandages	332 345	8
	12 cm	5x2 bandages	332 346	6
	15 cm	5x2 bandages	332 347	5
	20 cm	5x2 bandages	332 348	3
<u>Plastrona Slabs</u> 20 m length	10 cm	1 piece	335 935	2
	12 cm	1 piece	335 936	2
	15 cm	1 piece	335 937	2
	20 cm	1 piece	335 938	2
<u>Plastrona Large Slabs</u> rolls of 5 m length	40 cm	1 piece	335 360	20
	60 cm	1 piece	335 350	9