

Low-tack synthetic casting tape with excellent handling features:



Rhena® cast

Rhena cast is a high quality synthetic casting tape made of fibreglass with impregnated polyurethane resin. Rhena cast can be used for total immobilisation.

Description

- ▶ Due to the special polyurethane resin, Rhena cast can be moulded optimally with almost no adhesion to gloves. When Rhena cast is moulded, the surface becomes slightly foamy, contributing to the perfect lamination of each layer.
- ▶ The support material of Rhena cast is highly stretchable, enabling a crease-free cast and reducing the risk of constriction. The edges of Rhena cast bandage are clean cut and smooth, which is convenient for cast technicians and patients alike.
- ▶ Normally, 3 – 6 layers are required for immobilising limbs efficiently. Rhena cast becomes robust and load-bearing after 30 min.
- ▶ Rhena cast should ideally be stored at room temperature (between 15 °C and 30 °C). Storage at higher or lower temperatures can result in a reduced shelf life and sticky handling characteristics.



The excellent formation and lamination ensure optimal application and convenience for cast technicians and patients alike.

Presentations

		Unit	Code No.	Units per Case
3.6 m length white	3.8 cm	1 bandage	339 030	10
	5 cm	1 bandage	339 031	10
	7.5 cm	1 bandage	339 032	10
	10 cm	1 bandage	339 033	10
	12.5 cm	1 bandage	339 034	10
blue	5 cm	1 bandage	339 035	10
	7.5 cm	1 bandage	339 036	10
	10 cm	1 bandage	339 037	10
	12,5 cm	1 bandage	339 038	10
red	5 cm	1 bandage	339 039	10
	7.5 cm	1 bandage	339 040	10
	10 cm	1 bandage	339 041	10
	12.5 cm	1 bandage	339 042	10